GEBUIKERSHANDLEIDING DYNAMISCHE POSITIONERINGS ORTHESE (DPO) VAN REBACARE®

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USER MANUAL REBACARE® DYNAMIC POSITIONING ORTHOSIS (DPO)

- Before using the models, it is important to clean the outside with 1000 ppm chlorine, 70% alcohol, OxyFoam, OxyWipe or detergent. See also the guidelines of the Infection Prevention Working Group (WIP) (www.rivm.nl)
- Ask your occupational therapist or physiotherapist how to correctly position the models.
- Before positioning the models, it is important to evenly divide the stuffing. By correctly positioning and passing a hand (eventually with Handy) between the patient and the model, excess air can be drawn out through the filters. The filters will close automatically. This creates an individual mould of the body. The patient is supported comfortably and completely, without overpressure.
- The models cannot be washed in a washing machine or tunnel washer.
- For an explanation of the different applications, please refer to our website: www.rebacare.com/instructions
- The models are not suitable for children aged 3 and under.
- Our products do not contain any latex and can be disposed of with regular household waste.
- Please ensure that the patient indicates that the relevant cushion feels comfortable!
- Intended use: positioning users/patients (over 3 years old) to prevent and/or treat decubitus in the following indication(s): paraplegia, CVA, ALS, MS, Parkinson's disease, congenital abnormalities, comfort, dementia and/or conditions where postural management is desired, for example in Intensive Care Units (ICU), nursing homes, hospice or at home.

- MEDICAL PRODUCT CAN BE REUSED AFTER CLEANING
- CHECK THE PRODUCT FOR ANY DAMAGE
- DISPOSE OF THE PRODUCT IF DAMAGED
- DISINFECT WITH 70% ALCOHOL AND MAX. 1000 PPM CHLORINE, OXYFOAM, DETERGENT OR OXYWIPE
- WIPE THE PRODUCT WITH A CLOTH MOISTENED WITH WATER

REBACARE® GIVES A 2-YEAR PRODUCT GUARANTEE

After a two-year period of intensive use in terms of cleaning and hygiene (incontinence), REBACARE® recommends that the product is regularly inspected and replaced if necessary.

COVERS (DPO) FROM REBACARE®

REBACARE® gives a 6-month guarantee on the covers. These covers must be washed before use in accordance with the washing instructions. REBACARE® has 2 different matching covers:

- Standard Nicky stretch covers (80% cotton and 20% polyester, can be washed at 60 degrees Celsius) can be put in the dryer
- Stretch covers (100% cotton, can be washed at 60 degrees Celsius) can be put in the dryer

- WARNING:**
- Due to possible leakage of the contents (polystyrene beads and/or siliconized hollow fiber) to the environment, breathing problems may occur in the event of possible oral or nasal ingestion. To prevent this, only the REBACARE® models may be used with the original REBACARE® covers. You can order these original covers.
 - Make sure that the patient (if possible) is in a 30-degree position when lying on his side. This prevents the patient from turning, so that unwanted prone positioning is prevented.
 - The head models contain polyurethane ticking. Therefore they are not suitable for people with epilepsy.
 - Any serious incident related to the patient device must be reported to REBACARE® and the competent authority of the member state where the patient resides.
 - Do not combine parts with other positioning systems.
 - Any wound treatment of the patient should be performed before using the models.

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