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Foto 1

Foto 2

MANUAL OF THE ORTHOPEDIC BANDAGE SHOE / HEEL PROTECTOR BY REBACARE®

Orthopedic Bandage shoe /heel protector is meant:

- For preventive and curative use for patients with a risk of getting heel pressure ulcers (decubitus)
- To bridge the time during which patients cannot wear their own shoes because of wounds, phlebology or lymphatic disorders.
- The REBACARE® Orthopedic Bandage shoe / heel protector has a shortened life when used in the dryer.
- The bandage shoe / heel protector is a personal product. Washing the shoe at 60°C will ensure a long lifespan. Please close Velcro before washing.
- **Washable at 95 degrees** with a shortened life of about 25 washes. And with that a multi-use product. Please close Velcro before washing.
- The name of the patient can be written on the label using a ballpoint.
- The inlay sole (can be delivered upon request) can be taken out of the shoe and can be adjusted to the right size with a pair of scissors.
- Before washing the bandage shoe / heel protector, the Velcro should be closed.
- The leg should be placed in the bandage shoe / heel protector as shown in picture 1.
- Make sure that the grey sole of the bandage shoe / heel protector is placed on the outside of the shoe to ensure the anti-slip function.
- The lower bands should be placed on the instep of the foot. The right band should be brought to the left. Subsequently, the left band should be put over the right band, as exhibited on picture 2.
- Place the upper bands on the front of the shin. The left band should be brought to the right. Then, put the right band over the left band, as pictured in the 2nd photograph.
- Watch our useful instruction video: www.rebacare.com.

REBACARE® GIVES A 6 MONTH PRODUCT WARRANTY

After intensive use for a period of 6 months in terms of cleaning and hygiene (exudate), REBACARE® recommends that the product is regularly inspected and replaced if necessary.

Orthopedic
Bandage shoe /
heel protector

One size fits all (Size 3.0/10.0)

⚠️ WARNING:

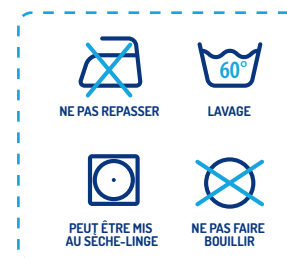
- **Oral or nasal ingestion of the model contents (siliconised hollow fibre) may cause respiratory problems.**
- **Any serious incident related to the patient device must be reported to REBACARE® and the competent authority of the member state where the patient resides.**
- **Any wound treatment of the patient should be performed before using the models.**
- **Make sure that the patient gives an indication of whether the bandage shoe / heel protector is comfortable!**

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