## GEBRUIKERSHANDLEIDING DYNAMISCHE POSITIONERINGS ORTHESE FREE EAR® OORKUSSEN VAN REBACARE®

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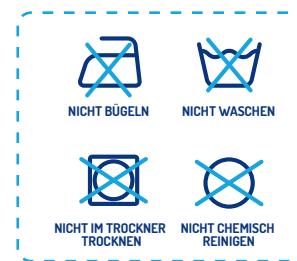
## USER INSTRUCTIONS FOR THE DYNAMIC POSITIONING ORTHOSIS FREE EAR® EAR PILLOW BY REBACARE®

The innovative Free Ear® ear pillow by REBACARE® is patented and leaves the ear free.

- Before using the Free Ear® ear pillow, it is important to clean the outside with 1000 ppm chlorine, 70% alcohol, OxyFoam, OxyWipe or detergent. See also the guidelines of the Infection Prevention Working Group (WIP) ([www.rivm.nl](http://www.rivm.nl))
- The Free Ear® ear pillow can not be washed in a washing machine or tunnel washer.
- For an explanation of the different applications, please refer to our website: [www.rebacare.com/instructions](http://www.rebacare.com/instructions)
- The Free Ear® ear pillow is not suitable for children aged 3 and under.
- The Free Ear® ear pillow does not contain any latex and can be disposed of with regular household waste.
- Intended use: leaves the ear free of users/patients (over 3 years old) to prevent/treat pressure ulcers in the following indications: Ear decubitus, Chondrodermatitis nodularis helicis, Oxygen tube over the ear, Tube-feeding or mask over the face, Hearing aid and Tube prone pillow during ventilation.
- Place the patient's head on the Free Ear® ear pillow so that the ear is positioned in the cut-out area.
- In the image of the Free Ear® ear pillow shown here (see image) the cushion is suitable for the left ear.
- To use the Free Ear® ear pillow of the patient for the right ear, the Free Ear® ear pillow has to be turned from right to left.
- Ensure that the face of the patient is directed toward the cut-out part.
- The patient's neck is on the narrowest part of the Free Ear® ear pillow.
- The Free Ear® ear pillow is not secured, but is loose on the bed.
- REBACARE® recommends changing the position of the patient every 3 hours, and if there are no medical objections, to turn the patient over so that the other ear can be positioned on the Free Ear® ear pillow.

Free Ear® ear pillow

REBACARE®  
Sasker van Heringawei 12  
9034 GD Marsum

The Netherlands  
[info@rebacare.com](mailto:info@rebacare.com)  
[www.rebacare.com](http://www.rebacare.com)

 MEDICAL PRODUCT CAN BE REUSED AFTER CLEANING

 CHECK THE PRODUCT FOR ANY DAMAGE

 DISPOSE OF THE PRODUCT IF DAMAGED

 DISINFECT WITH 70% ALCOHOL AND MAX. 1000 PPM CHLORINE, OXYFOAM, DETERGENT OR OXYWIPE

 WIPE THE PRODUCT WITH A CLOTH MOISTENED WITH WATER

DO NOT IRON

DO NOT PUT IN WASHER

DO NOT PUT IN DRYER

DO NOT DRY-CLEAN

### REBACARE® GIVES A 2-YEAR PRODUCT GUARANTEE

After a two-year period of intensive use in terms of cleaning and hygiene (exudate), REBACARE® recommends that the product is regularly inspected and replaced if necessary.

**REBACARE® gives a 6-month guarantee on the covers. These covers must be washed before use in accordance with the washing instructions. REBACARE® has 3 different matching covers:**

**Standard Nicky stretch covers (80% cotton and 20% polyester, can be washed at 60 degrees Celsius)**

**Stretch covers (100% cotton, can be washed at 60 degrees Celsius)**

**Shrink-free covers without stretch (100% cotton, can be washed at 95 degrees Celsius)**

### WARNING:

- **Oral or nasal ingestion of the model contents (polystyrene beads and/or siliconised hollow fibre) may cause respiratory problems. When repositioning the patient, ensure they are lying in a 30-degree lateral position (if possible). This prevents the patient turning too far and unintentionally ending up in a prone position.**
- **Due to potential leakage of the contents, REBACARE® models may only be used with the original REBACARE® covers. These original covers are not included and can be ordered separately.**
- **The product for the head is not suitable (contains polyurethane ticking). Therefore they are not suitable for people with epilepsy.**
- **Any serious incident related to the patient device must be reported to REBACARE® and the competent authority of the member state where the patient resides.**
- **Any wound treatment of the patient should be performed before using the product.**

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